

Surface Disinfection

Using CaviCide: Spray ▶ Wipe ▶ Spray

Clean before disinfecting



1. Spray CaviCide directly onto surface to be cleaned.



2. Wipe with clean paper towel to remove debris and bioburden.



3. Discard towel.

Disinfect after cleaning



4. Spray CaviCide directly onto precleaned surface, thoroughly wetting area. Follow label instructions for appropriate contact times.



5. Wipe with clean paper towel.



6. Discard towel.

Use CaviCide® to clean and disinfect:

Potentially contaminated noncritical instruments and equipment

Hard, nonporous surfaces or items to prevent cross contamination, such as:

- Light fixtures
- Clinical and laboratory surfaces
- Telephone and office machines

Note: Follow standard precautions for personal protection as appropriate.