

Surface Disinfection

Using CaviWipes® is an Easy 2 Step Process

STEP 1: Clean before disinfecting



A. Dispense CaviWipes towelette.



B. Clean surface with towelette to remove debris and bioburden.



C. Discard used towelette.

STEP 2: Disinfect after cleaning



A. Dispense another CaviWipes towelette.



B. Disinfect pre-cleaned surface with towelette. Follow label instructions for appropriate contact times.



C. Discard used towelette.

To prevent cross contamination, use CaviWipes to clean, disinfect and decontaminate exterior, hard, non-porous surfaces of:

- Infant incubators, cribs and warmers
- Anesthesia machines and respiratory therapy equipment
- Laboratory equipment
- Ambulance equipment
- Tables, chairs and workstations
- IV poles, bed railings and handrails
- Bathroom fixtures
- Telephones

Note: Follow standard precautions for personal protection as appropriate.

CaviWipes Family

